**Clo-Sur<sup>PLUS</sup> P.A.D.<sup>TM</sup>**  
**Antimicrobial Barrier**  
**Instructions for Use**

Scion Cardio-Vascular, Inc.  
14256 S.W. 119<sup>th</sup> Avenue  
Miami, FL 33186 USA  
Phone: 305-259-8880  
Toll Free: 1-866-259-8881  
Fax: 305-259-8878

Emergo Europe  
Molenstraat 15  
2513 BH The Hague  
The Netherlands  
Phone: +31 (0) 70 345-8570  
Fax: +31 (0) 70 346-7299

P/N 00639 Rev. E

**ENGLISH**

**Clo-Sur<sup>PLUS</sup> P.A.D.<sup>1M</sup> Antimicrobial Barrier**  
Instructions for Use

**DESCRIPTION:**

The Scion Cardio-Vascular *Clo-Sur<sup>PLUS</sup> P.A.D.* is a soft, non-woven topical hemostasis pad. The *Clo-Sur<sup>PLUS</sup> P.A.D.* is designed to provide a barrier to microbial penetration and has demonstrated in-vitro antibacterial activity that prevents colonization of the dressing. *Clo-Sur<sup>PLUS</sup> P.A.D.* is packed in a foil pouch and sterilized by E-beam radiation.

**INDICATIONS:**

The Scion Cardio-Vascular *Clo-Sur<sup>PLUS</sup> P.A.D.* is intended for use in the local management of bleeding wounds and to provide a barrier to bacterial penetration of the dressing in all patients and for the promotion of rapid control (hemostasis) of bleeding in patients following hemodialysis and for those on anticoagulation therapy. The dressing is indicated for the following wounds: skin puncture sites for vascular procedures, percutaneous catheters or tubes.

**CONTRAINDICATIONS:**

- ◆ No known contraindications.

**WARNINGS:**

READ ALL INSTRUCTIONS PRIOR TO USE. OBSERVE ALL WARNINGS AND PRECAUTIONS NOTED THROUGHOUT THESE INSTRUCTIONS. FAILURE TO DO SO MAY RESULT IN COMPLICATIONS.

- ◆ Do not reuse. Discard after one procedure.
- ◆ Product reuse is not feasible and could be unsafe due to risk of cross-infection. The pad material dissolves once it comes in contact with bodily fluids. The product cannot be effectively cleaned, decontaminated, or reprocessed into its original form after a single application.

**PRECAUTIONS:**

- ◆ Do not use open or damaged packages.
- ◆ Product is water soluble. Do not autoclave.
- ◆ Do not expose to organic solvents.

**STORAGE INFORMATION:**

- ◆ Store in a temperature controlled environment (15° - 30° C)
- ◆ Keep dry, protect from light, and avoid prolonged exposure to high temperatures.

**COMPLICATIONS:**

Some of the complications are, but not limited to bleeding, hematomas, pseudoaneurysm, rash and swelling.

**PROCEDURE:**

1. Apply pressure proximal to treatment area.
2. Verify treatment area is dry prior to applying the P.A.D.
3. Apply the **Clo-Sur<sup>PLUS</sup> P.A.D.** over the treatment area and place a clean, sterile gauze pad over the **Clo-Sur<sup>PLUS</sup> P.A.D.**
4. Gradually release proximal pressure to permit blood to contact the **Clo-Sur<sup>PLUS</sup> P.A.D.**
5. Reapply proximal pressure, as required, to control bleeding.
6. Verify hemostasis by gradually releasing pressure and checking dressing for additional bleeding. Do not lift P.A.D. to verify.
7. Once hemostasis is achieved, secure the treatment area with the **Clo-Sur<sup>PLUS</sup> P.A.D.** left in place using a dry adhesive dressing.

**POST PROCEDURE INSTRUCTIONS FOR Clo-Sur<sup>PLUS</sup> P.A.D.**

1. After 24 hours, remove dressing, wet **Clo-Sur<sup>PLUS</sup> P.A.D.** with water and allow it to dissolve and remove slowly.
2. Observe treatment area for any signs of infection, including drainage, swelling, redness, increased pain, etc. Watch for signs of hematoma development, which will be a hard painful bruise at the treatment area.
3. If a hematoma or bleeding develops, hold pressure at the site, call your physician immediately or go to the emergency room.
4. Call your physician's office if you have any problems or concerns.

*There is no expressed or implied warranty, including merchantability and fitness for a particular purpose for the **Clo-Sur<sup>PLUS</sup> P.A.D.** Under no circumstances shall Scion Cardio-Vascular be liable for any direct, incidental or consequential damages other than as expressly provided by specific law. No person has the authority to bind Scion Cardio-Vascular to any representation or warranty except as specifically set forth herein. Descriptions or specifications are meant to generally describe the **Clo-Sur<sup>PLUS</sup> P.A.D.** and related procedures, and do not constitute any expressed warranties or practice.*

**Clo-Sur<sup>PLUS</sup> P.A.D.** is a trademark of Scion Cardio-Vascular, Inc. All rights reserved.

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